

Hip FAI burrs to match
your range of **sculpting needs**



The right tools for the right job

DYONICS[®] High Visibility Sheath Hip Burrs – from the leader in hip arthroscopy.*


Our expanding line of new hip burrs gives surgeons the clinical performance to address a range of procedural needs and the capability to confidently perform a range of procedures, from aggressive bone removal to finer bone sculpting. Each burr features an outer, narrow-neck sheath that improves visualization of the surgical site. Just a few more reasons to use the industry's flagship brand for resection.



1 | DYONICS High Visibility Sheath

4.0 and 5.5 mm Abrader Burr (72203128, 72203127)

The DYONICS High Visibility Sheath Abrader Burr lets surgeons see more anatomical landmarks, which can help to minimize inadvertent over-resection.

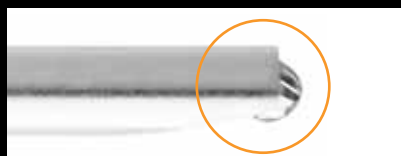

Conventional Abrader Burr



DYONICS High Visibility Sheath Abrader Burr

- Compared to a conventional abrader burr (left), the narrow-neck design on the new DYONICS High Visibility Sheath Abrader Burr (right) allows you to see more of the burr and the anatomy.
- Offers the same familiarity and versatility of a traditional Abrader burr, while still protecting the labrum.
- The high visibility outer sheath edges have been rounded to help minimize damage to the surrounding tissues.

* 2010 Market Data



2 | DYONICS High Visibility Sheath

4.0 and 5.5 mm Flat Top Burr ([72203130](#), [72203129](#))

The unique flat-top or end-cut design facilitates the ability to sculpt offending acetabular overhang, leaving a flat surface while still protecting the labrum.



- The rounded perimeter makes this a versatile burr for other procedures, such as arthroscopic femoral osteochondroplasty, and spur removal in the peritrochanteric space.

3 | DYONICS High Visibility Sheath

4.0 and 5.5 mm Barrel Burr ([72203132](#), [72203131](#))

The Barrel Burr has a unique aggressive side-cutting, and gentle end-cutting, transitional flute design for refined resection in hip anatomy.



- 6 side-cutting flute design provides aggressive resection for osteochondroplasty and other procedures where rapid bone removal is required.
- 12 end-cutting flute design allows precise sculpting for acetabuloplasty like a round abrader burr.



Ordering Information

Reference #	Description
72203127	High Visibility Sheath 5.5 mm Abrader Burr
72203128	High Visibility Sheath 4.0 mm Abrader Burr
72203129	High Visibility Sheath 5.5 mm Flat Top Burr
72203130	High Visibility Sheath 4.0 mm Flat Top Burr
72203131	High Visibility Sheath 5.5 mm Barrel Burr
72203132	High Visibility Sheath 4.0 mm Barrel Burr

Smith & Nephew, Inc.
150 Minuteman Road
Andover, MA 01810 USA

www.smith-nephew.com
T +1 978 749 1000
US Customer Service: +1 800 343 5717
International Customer Service: +1 978 749 1140

*Trademarks of Smith & Nephew. Certain marks
Registered US Patent and Trademark Office.
©2011 Smith & Nephew, Inc. All rights reserved.
Printed in USA. 10/11 10600704 Rev. C