


Hip Reference Guide

 **smith&nephew**
Q-FIX[®] CURVED XL
All-Suture Anchor

Supporting healthcare professionals



1. Expose the bone at the desired insertion location for the implant. Attach the appropriate drill bit to a standard orthopedic drill. Be sure to chuck the drill no farther than the flats on the proximal end of the drill.

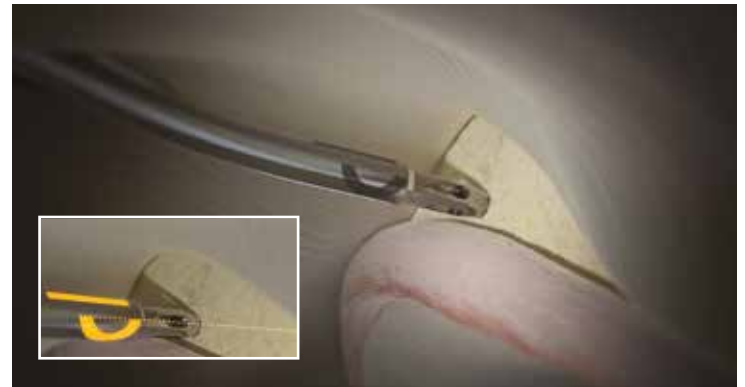

2. Position the tip of the Drill Guide with Obturator against the bone surface at the desired position then remove the Obturator. The crescent laser mark indicates the direction of the curve, while the vertical laser mark helps project the trajectory of the drill bit.



3. Insert the drill into the drill guide and drill the bone hole. Advance the drill until the drill bottoms out in the drill guide.


Caution: Do not use the drill without the drill guide as the bone surface can be damaged. Confirm that bone hole is clear of bone debris.



4. Remove the drill and insert the implant through the drill guide into the bone hole. Be sure that the implant inserter is fully engaged with the drill guide throughout the deployment of the anchor.



5. Rotate the activation knob on the proximal end of the inserter clockwise. Rotate the knob until a hard stop is reached and the suture cleat is fully exposed.

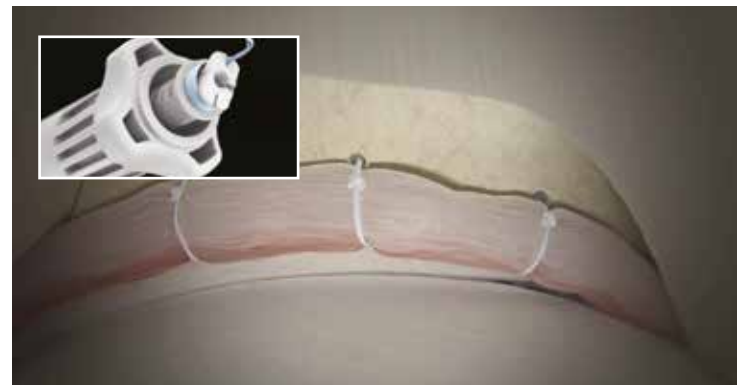

6. Release the sutures from the cleat and remove the inserter and drill guide. If placing more than one implant, repeat steps 2 through 5 to complete repair.

Ordering information

Q-FIX° All-Suture Anchors

1.8mm Q-FIX MINI All-Suture Anchor

Reference #	Description
Anchors:	
72290123	1.8mm Q-FIX MINI All-Suture Anchor
Reusable Instruments:	
25-1812	1.8mm Reusable PATHFINDER° Obturator
Disposable Kits:	
72290125	1.8mm Q-FIX MINI Disposable Kit
72290126	1.8mm Q-FIX MINI XL Disposable Kit

1.8mm Q-FIX All-Suture Anchor

Reference #	Description
Anchors:	
25-1800	1.8mm Q-FIX All-Suture Anchor
Reusable Instruments:	
72290119	1.8mm Reusable Curved Drill Guide, XL
72290120	1.8mm Reusable Curved Drill Guide
25-1812	1.8mm Reusable PATHFINDER Obturator
Disposable Kits:	
25-1810	1.8mm Q-FIX Disposable Kit
25-1811	1.8mm Q-FIX Disposable Kit XL
Disposable Accessories:	
72290118	1.8mm Disposable Flex Drill

2.8mm Q-FIX All-Suture Anchor

Reference #	Description
Anchors:	
25-2800	2.8mm Q-FIX All-Suture Anchor
Reusable Instruments:	
25-2812	2.8mm Reusable PATHFINDER Obturator
25-2820	2.8mm Reusable Bone Punch for Q-FIX All-Suture Anchor
Disposable Kits:	
25-2810	2.8mm Q-FIX Disposable Kit

All photos and diagrams are for illustrative purposes only. Products not to scale.

For more information on the application of any products discussed in the presentation, as well as indications for use, contraindications, and product safety information, please consult the Instructions for Use (IFU) for such product. To order these instruments, call **+1 800 343 5717** in the U.S. or contact an authorized Smith & Nephew representative.

Supporting healthcare professionals for over 150 years

Manufactured by:
ArthroCare Corporation
7000 West William Cannon Drive
Austin, TX 78735 USA

Distributed by:
Smith & Nephew, Inc.
150 Minuteman Road
Andover, MA 01810 USA

www.smith-nephew.com
T +1 978 749 1000
US Customer Service:
+1 800 343 5717

°Trademark of Smith & Nephew.
©2018 Smith & Nephew.
All rights reserved. Printed in USA.
11939 V1 01/18

