


Shoulder Reference Guide

 **smith&nephew**
SUTUREFIX
CURVED
All-Suture Anchor
Supporting healthcare professionals

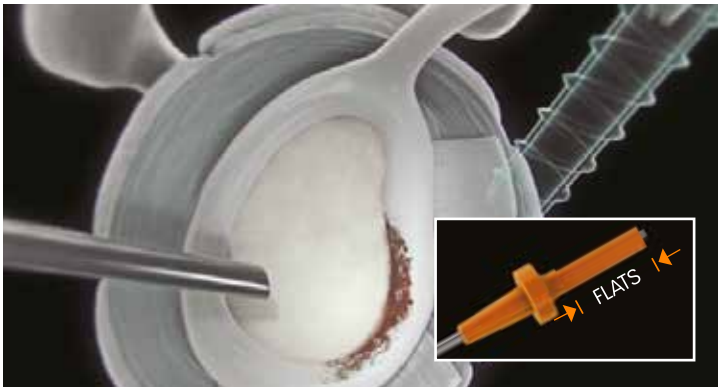

1. Expose the bone at the desired insertion location for the implant. Attach the appropriate drill bit to a standard orthopedic drill. Be sure to chuck the drill no farther than the flats on the proximal end of the drill.



2. Position the tip of the Drill Guide with Obturator against the bone surface at the desired position then remove the Obturator. The crescent laser mark indicates the direction of the curve.



3. Insert the drill into the drill guide and drill the bone hole. Advance the drill until the drill bottoms out in the drill guide.

Caution: Do not use the drill without the drill guide as the bone surface can be damaged.



4. Remove the drill and insert the implant through the drill guide into the bone hole. Be sure that the anchor inserter is fully engaged with the drill guide prior to anchor deployment.



5. Depress the green button to unlock the sliding ring. Pull back on the sliding ring until a click is heard, confirming proper anchor deployment.



6. Unwind the suture from the cleat, and slowly pull back on the inserter to remove. Then, remove the guide. If placing more than one implant, repeat steps 2 through 5 to complete repair.

Ordering information

SUTUREFIX CURVED All-Suture Anchor and Guide System	
Reference #	Description
SUTUREFIX CURVED Suture Anchors	
72204687	SUTUREFIX CURVED Anchor 1.7mm with one ULTRABRAID° #2 Suture (Blue)
72204688	SUTUREFIX CURVED Anchor 1.7mm with one ULTRABRAID #2 (COBRAID Blue)
72204689	SUTUREFIX CURVED Anchor 1.9mm with two ULTRABRAID #1 Sutures
72205060	SUTUREFIX CURVED XL Anchor 1.7mm with one ULTRABRAID #2 Sutures (Blue)
72205061	SUTUREFIX CURVED XL Anchor 1.7mm with one ULTRABRAID° #2 Suture COBRAID-blue)
Drill Guides, Drills and Obturators for SUTUREFIX CURVED Suture Anchors	
72204690	SUTUREFIX CURVED 1.7mm, Shoulder Flexible Drill
72204691	SUTUREFIX CURVED 1.9mm, Shoulder Flexible Drill
72204692	SUTUREFIX CURVED Shoulder Drill Guide, Crown tip, Reusable
72204694	SUTUREFIX CURVED Shoulder Obturator, Reusable
72205062	SUTUREFIX CURVED 1.7mm, XL Flexible Drill
72205063	SUTUREFIX CURVED XL Drill Guide, Crown tip
72205064	SUTUREFIX CURVED XL Obturator
72205141	SUTUREFIX Inline Drill Guide for Straight and Flexible Devices
72205142	SUTUREFIX Inline Drill Guide for Straight and Flexible Devices, XL Length
SUTUREFIX ULTRA XL Suture Anchors	
72203841	SUTUREFIX ULTRA Anchor 1.7mm XL with one (#2) ULTRABRAID° Suture (Blue)
Drill Guides, Drills and Obturators for SUTUREFIX ULTRA XL Suture Anchors	
72204669	SUTUREFIX ULTRA Drill XL 1.7 mm, Disposable
72203844	Crown Tip Drill Guide XL
72203845	Spike Tip Drill Guide XL
72203846	Blunt Tip Obturator XL
72203899	Cannulated Obturator XL

All photos and diagrams are for illustrative purposes only. Products not to scale.

For more information on the application of any products discussed in the presentation, as well as indications for use, contraindications, and product safety information, please consult the Instructions for Use (IFU) for such product. To order these instruments, call **+1 800 343 5717** in the U.S. or contact an authorized Smith & Nephew representative.

Supporting healthcare professionals for over 150 years

Manufactured by:
ArthroCare Corporation
7000 West William Cannon Drive
Austin, TX 78735 USA

Distributed by:
Smith & Nephew, Inc.
150 Minuteman Road
Andover, MA 01810 USA

www.smith-nephew.com
T +1 978 749 1000
US Customer Service:
+1 800 343 5717

°Trademark of Smith & Nephew.
©2018 Smith & Nephew.
All rights reserved. Printed in USA.
12134 V1 01/18