BIOSTEP[◇] Ag

Collagen Matrix Dressing
with Silver

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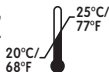
CAUTION: Federal (USA) law restricts this device to
sale by or on the order of a physician.

STERILE R

ENG

BIOSTEP® Ag Collagen Matrix Dressing with Silver is an advanced wound care dressing composed of collagen, sodium alginate, carboxyl methylcellulose, ethylenediaminetetraacetic acid (EDTA) and silver chloride.

Properties and mode of action

BIOSTEP Ag Collagen Matrix Dressing is a topically applied wound dressing designed to transform into a soft gel sheet on contact with wound exudate. It maintains a moist wound environment at the wound surface that aids in the formation of granulation tissue and epithelialization. The EDTA in the dressing binds the zinc which is essential to matrix metalloproteinases (MMP) function. The removal of zinc inhibits the activity of the MMPs creating a suitable environment for wound healing. The dressings act as an effective barrier to bacterial and fungal penetration. The silver content is intended to prevent colonization of the dressing. The dressings may be cut to shape and layered for the management of deep wounds.

Indications for Use

BIOSTEP Ag is indicated for management of full and partial thickness wounds including: pressure ulcers, diabetic ulcers, ulcers caused by mixed vascular aetiologies, venous ulcers, donor and graft sites, abrasions, traumatic wounds healing by secondary intention, dehisced surgical wounds, first and secondary degree burns.

Directions for use

- Prior to application, the wound bed should be debrided and then irrigated with an appropriate wound cleanser or water. The use of saline could affect the efficacy of the silver.
- The dressing should be cut to fit the wound

size exactly. For heavily exuding wounds, the dressing should be applied directly to the wound bed. For dry wounds with minimal exudate, the wound bed should be moistened with water to begin the gelling process.

- BIOSTEP Ag should be covered with a suitable secondary dressing.
- Dressings may be left in place for up to 6 days depending on the level of exudate or as good nursing practice dictates.

Dressing Removal

- Remove secondary dressing with care. Gently remove BIOSTEP Ag Collagen Matrix Dressing with Silver ensuring that the dressing removal does not damage delicate newly-formed tissues.
- Cleanse the wound site using an appropriate wound cleanser or water prior to application of a new BIOSTEP Ag as per application instructions.

Contraindications

BIOSTEP Ag Collagen Matrix Dressing with Silver should not be used on patients with a known allergy or sensitivity to porcine collagen, to silver, any other ingredient or on third degree burns.

Active Ingredient

BIOSTEP Ag contains silver chloride whose purpose is to reduce the incidence of bacterial colonization within the wound dressing.

Precautions

- If sensitivity to BIOSTEP Ag develops, discontinue use.
- BIOSTEP Ag dressings may be used under compression therapy under the supervision of a health care professional.
- United States' federal law restricts this device to sale by or on the order of a physician.

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Trademark of/Marque